## Hydrofera Blue Ready foam



Ready. Set. Go.

Simply set in place and secure; no hydration required.

## Use When:

- Antibacterial properties are needed to help prevent infection
- Absorption of moderate to heavy levels of exudate is required
- Compression therapy is needed
- Total contact casting is needed

## Application:

- Simply place and secure; no hydration required
- Secondary dressing not required
- Tape or wrap required for fixation

## **Wound Site Preparation**



Cleanse the wound bed in accordance with established procedures.

Secure



Secure the dressing using a bandage or tape.

**Hydrofera Blue Ready** foam can be used in conjunction with compression wraps and total contact casting.

## Select Size and Apply



Select the appropriate dressing size to ensure the dressing will cover the entire wound. Apply with the foam side towards the wound, and the shiny side with the thin film up.

### Wear Time



Dressing can be left in place up to 7 days, depending on the exudate level and clinical conditions of the wound.

# Hydrofera Blue Ready foam Ordering Information Stock No Product Size Quantity/Box

Product Size	Quantity/Box	HCPCS
2.5" x 2.5" (6.4 cm x 6.4 cm)	10	A6209
4" x 5" (10.2 cm x 12.7 cm)	10	A6210
8" x 8" (20.3 cm x 20.3 cm)	10	A6211
	2.5" x 2.5" (6.4 cm x 6.4 cm) 4" x 5" (10.2 cm x 12.7 cm)	2.5" x 2.5" (6.4 cm x 6.4 cm) 10 4" x 5" (10.2 cm x 12.7 cm) 10

For more information, see product instructions included with the product.

R Caution: Federal (USA) law restricts this device for sale or on the order of a physician or licensed healthcare professional. Refer to Instruction for Use for contraindications, warnings, precautions and possible complications.

For product questions or detailed clinical questions concerning our products in the US call **1.888.740.8999.** In Canada call **1.800.263.7400.** 

The Hollister logo is a trademark of Hollister Incorporated. Hydrofera Blue Ready and the logo are trademarks of Hydrofera, LLC.

Made and printed in the USA.

©2014 Hollister Incorporated 922498-0514

Hydrofera**BLUE** 

Manufactured for **Hollister Incorporated** 2000 Hollister Drive Libertyville, IL 60048 USA

Distributed in Canada by: Distribué au Canada par: **Hollister Limited** 95 Mary Street Aurora, Ontario L4G 1G3

www.hollisterwoundcare.com

